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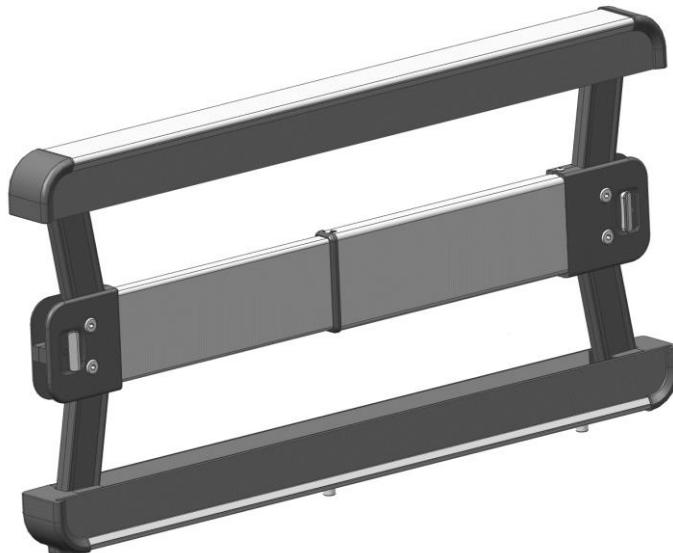
.**bock**™

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für

- **practico 25|80**
- **practico 25|80 plus**
- **livorno niedrig**
- **floorline 15|80**
- **practico alu**
- **practico alu plus**
- **floorline 9.5|80**
- **stellar**

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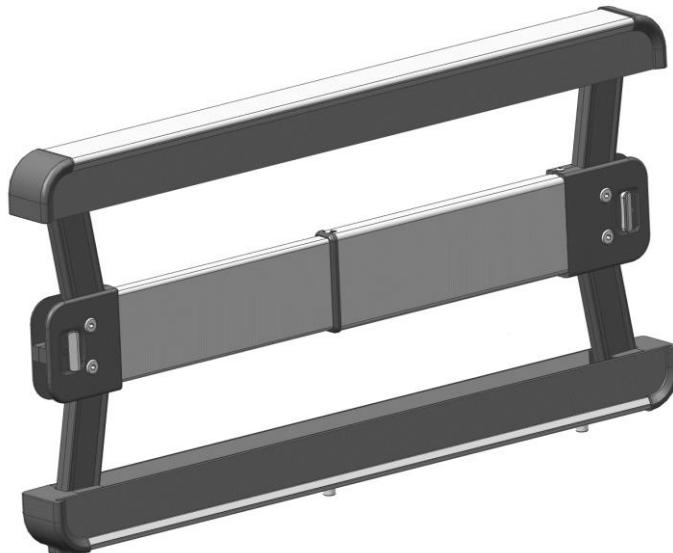
Dear valued customer,

With your decision to purchase a side rail from Hermann Bock GmbH, you will receive a durable care product with superior functionality at the highest safety level.

This product was designed with a focus on persons in need of care, whose confidence must be reinforced and whose life needs protection. With this health care product, we meet these requirements.

We urge you to prevent potential malfunctions and the risk of accidents by complying strictly with the safety and operating instructions and by carrying out the necessary maintenance.

A handwritten signature in black ink, appearing to read "Klaus Bock".

Klaus Bock

Table of contents

1	Preface and general instructions	4
1.1	Reference to the assembly and operation manual for the nursing care bed ...	4
1.2	Definition of the group of persons	4
1.3	Safety instructions.....	5
1.4	Service life / warranty	5
2	General description of the functions.....	6
2.1	Construction design and function	6
2.2	Bracket variants - steel frame and aluminium frame.....	6
2.3	Removal of the existing side rails	7
2.4	Hole overview for the different variants of the steel frame	8
2.5	Assembly of the SR1 side rail for nursing care beds with a steel frame	9
2.6	Assembly of the SR1 side rail for nursing care beds with an aluminium frame ..	
	11
2.7	Safety distances.....	14
2.8	Dismantling the SR1 side rail	14
2.9	Operating the SR1 side rail	15
2.10	Transport, storage and operating conditions.....	16
2.11	Disposal	16
3	Cleaning, maintenance and disinfection	16
4	Regular inspections with service.....	16

1 Preface and general instructions

This assembly and operation manual describes the assembly of the side rail system SR1 - for the defined nursing care beds of Hermann Bock GmbH and use of the SR1 side rail. Read these assembly instructions completely to avoid damage due to incorrect assembly and handling.

Assembly of the SR1 side rail is only to be carried out by qualified personnel and must under no circumstances be carried out by a private user.

The installation of the SR1 side rail is approved for the defined nursing care beds from the manufacturing date of January 2016. Preparations at the nursing care bed are given by a supplementary instruction.

This assembly and operation manual contains safety instructions. All persons who work with the nursing care beds must be familiar with the contents of these instructions. Improper operation can result in personal injuries.

1.1 Reference to the assembly and operation manual for the nursing care bed

The assembly and operation manual of the respective Hermann Bock GmbH nursing care bed must be observed.

Bock Note

If you have received this assembly and operation manual, then your product has additional equipment that is not described in the main assembly and operation manual. Therefore, please attach these instructions to the main assembly and operation manual.

1.2 Definition of the group of persons

Qualified personnel

Employees of the operator are referred to as qualified personnel. They are entitled to deliver the nursing care bed, assemble, dismantle and transport it, on the basis of their training or instructions. Besides knowing how to operate, assemble and disassemble the nursing care bed, these persons must be instructed according to the guidelines concerning the cleaning and disinfection of the nursing care bed.

1.3 Safety instructions

The intended use/operation of all moving parts is as important for the safety of the persons in need of care as well as for the relatives and the caregivers/nursing staff to avoid potentially dangerous situations. The prerequisite for this is the correct assembly and operation of the side rails. When operating the side rails, it is also essential to take into account the individual physical constitution of the person in need of care, as well as the type and extent of the disability.

These recommendations apply particularly:

- if the person in need of care or the caregivers could be at risk due to those accidental adjustments;
- if the side rails are in a raised position and there could be danger of trapping and crushing,
- if children are unsupervised in the room with the bed.

The side rails should only be operated by trained caregivers or relatives or in the presence of trained persons.

When adjusting the side rails, particular care must be taken to ensure that no limbs are located in the side rails within the adjustment range.

Please note that the safety instructions in the main assembly and operating manual for the nursing care bed also apply.

Hazards may arise when using the side rails on incompatible beds.

1.4 Service life / warranty

This side rail has been designed, engineered and manufactured to function safely over a long period of time. The service life depends on operating conditions and frequencies. Please refer to the main assembly and operating manual for your nursing care bed.

Caution: Unauthorised technical changes to the product voids all warranty claims.

This product is not approved for the North American market, particularly not for the United States of America (USA). Distribution and use of the nursing care bed in these markets, including through third parties, is prohibited by the manufacturer.

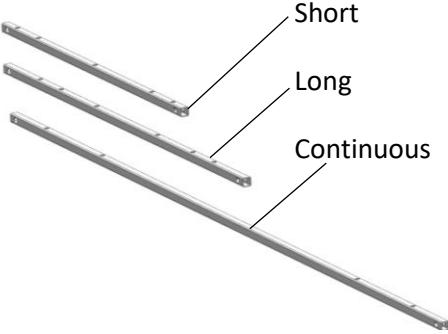
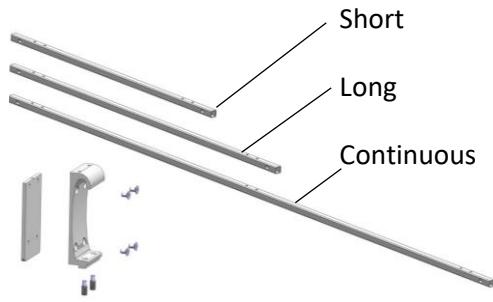
2 General description of the functions

2.1 Construction design and function

The SR1 side rail

All side rail variants serve to protect against falling out. The SR1 side rails allow safe and convenient entry and exit at the same time. By means of a simple release mechanism, each individual side rail element can be brought into a raised or lowered position.

2.2 Bracket variants - steel frame and aluminium frame

Bracket for nursing care beds with steel frame	Bracket for nursing care beds with aluminium frame
 <p>Short Long Continuous</p>	 <p>Short Long Continuous</p>
<p>Suitable for the following nursing care beds:</p> <ul style="list-style-type: none"> • pratico 25 80 • pratico 25 80 plus • livorno low • floorline 15 80 	<p>Suitable for the following nursing care beds:</p> <ul style="list-style-type: none"> • pratico alu • pratico alu plus • floorline 9.5 80 • stellar

2.3 Removal of the existing side rails

When removing the existing side rail, you must follow the steps for assembly given in the main assembly and operation manual in reverse order for the side rail. The further dismantling of the fittings is explained here. See also our homepage.

Move the nursing care bed to the highest position for easier dismantling.

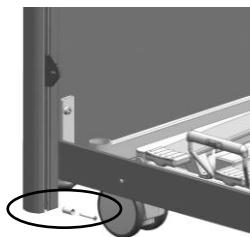
The existing side rails must be removed.

The fitting must also be removed from the C-rail.

Loosen the screw at the bottom of the C-rail, remove the fitting downwards and replace the screw connection.

Now the C-rail is prepared without a fitting and for the further assembly of the SR1 side rail.

2.4 Hole overview for the different variants of the steel frame

Head side

Foot side


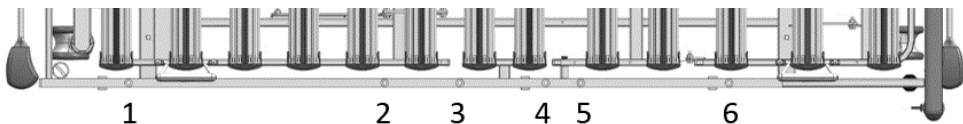
Figure	Variant	Position of the hole						
		1	2	3	4	5	6	
Head side	Foot side	SR1	1	2	3	4	5	6
		51%	X		X			
		60%/40%	X		X		X	X
		40%/60%	X	X		X		X
		Continous	X					X


2.5 Assembly of the SR1 side rail for nursing care beds with a steel frame

Example head side long

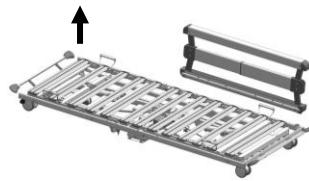Move the nursing care bed to the highest position for easier assembly.

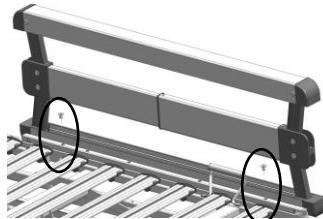Place the bracket with the side rail on the longitudinal frame as described in point 2.4 and slide it onto the existing holes, looking from above. Insert the sleeve nut M8 at the holes shown from above.


Now take the cylinder screws M8x55 to hand.

Screw the two cylinder screws M8x55 from below through the steel frame into the sleeve nut M8.


Tighten the screws.

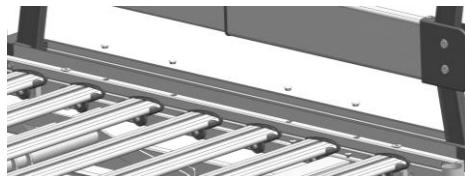Press the cover plugs into the open holes.

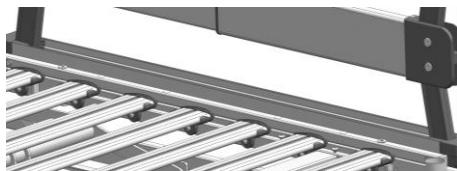
Now all holes should be closed and the SR1 is ready for operation.


Move the side rail to the upper position.

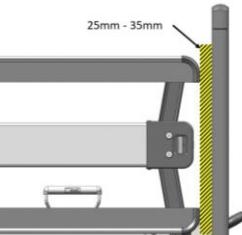Attention: The measurement between the vertical outer edge of the side rails and the nearest possible component must be between 25mm - 35mm horizontally, depending on the bed head.

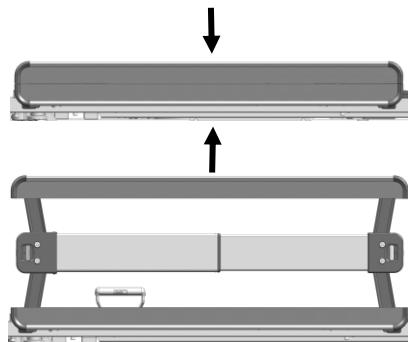
Functional check. Lower the side rails and put them back in place.


2.6 Assembly of the SR1 side rail for nursing care beds with an aluminium frame

Example head side long – preparation of nursing care bed

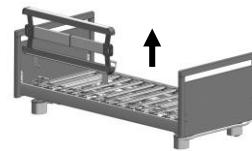Move the nursing care bed to the highest position for easier assembly.

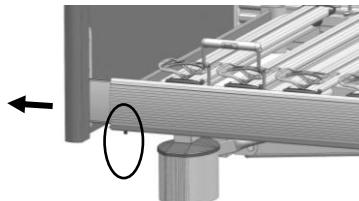
Loosen the bed extension on the foot side via the threaded rod / hand screw and pull out the bed extension slightly.


Now take the spacer to hand. Hook the spacer into the longitudinal frame from above.

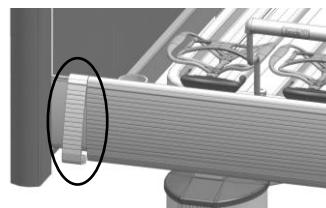Repeat the procedure on the other side.

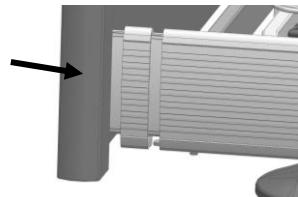
Then push the bed extension back together completely.

Tighten the threaded rods on both sides tightly again.

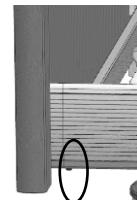The nursing care bed is now prepared for further assembly of the SR1 side rail.

Example head side long – preparation of SR1 side rail

Connect bracket and plate with the countersunk screws

The next step is to lightly screw the threaded rods into the aluminium bracket.

The next step is to mount the holder to the SR1 side rail using the pan-head screws.

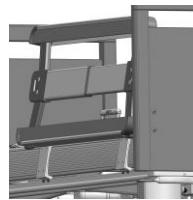
Follow the same procedure on the other side of the side rails. Now the SR1 side rail is prepared.


Example head side long – assembly of SR1 side rail

Take the entire SR1 side rail and hook it into the longitudinal frame from above.

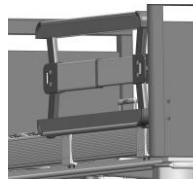
Raise the side rail to its uppermost position so that it is also correctly positioned on the aluminium frame.


The next step is to move the side rail.

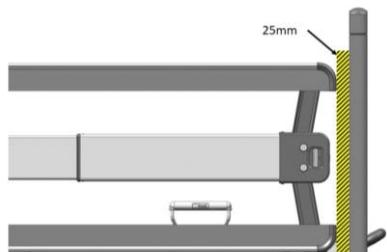
Be sure to maintain the safety distances in the following chapter.

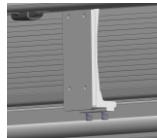Now screw the four threaded rods tightly to the entire side rail.

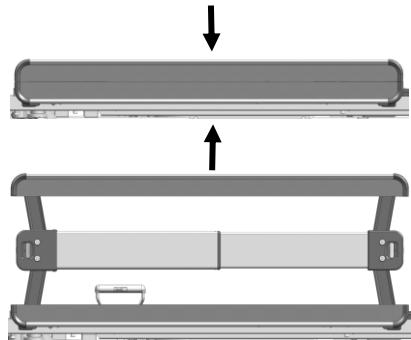
Functional check. Lower the side rails and put them back in place.


2.7 Safety distances

Attention: The measurement between the vertical outer edge of the side rail and the nearest possible component must

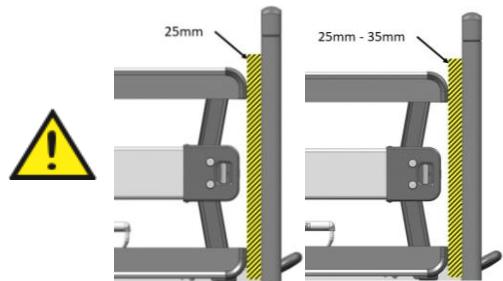- for the aluminium frame, be 25 mm horizontally.
- for the steel frame, be between 25mm - 35mm horizontally, depending on the bed head.

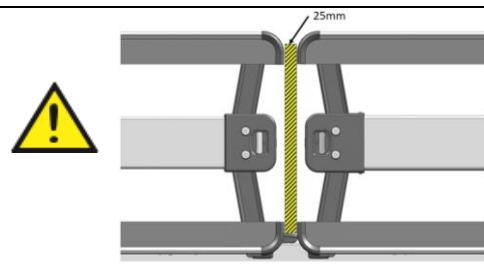
Attention: The measurement between the side rails must be 25mm horizontally.

Attention: The measurement between the vertical outer edge of the side rail and the nearest possible component must

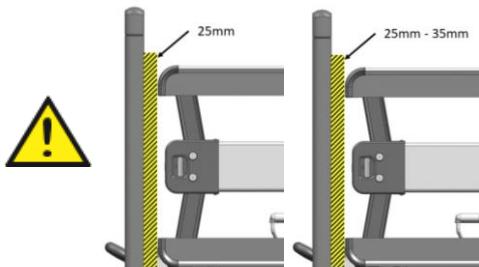- for the aluminium frame, be 25 mm horizontally.
- for the steel frame, be between 25mm - 35mm horizontally, depending on the bed head.


2.8 Dismantling the SR1 side rail

The dismantling of the side rail is conducted in the reverse order of the assembly description.

2.9 Operating the SR1 side rail


Each side rail element can be adjusted independently from the other.. The release button for the adjustment is located in the middle of the side rail.

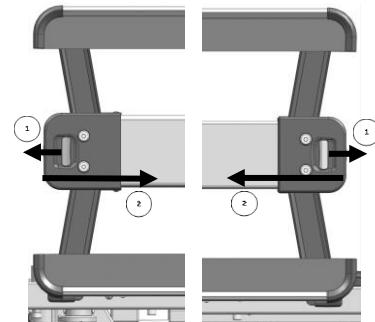To lower a side rail, grasp the release button with one hand and (1) pull it outward. Then, with the release held in, push the entire holder inwards a little (2) so that the latch is free. In the case of a continuous side rail, this release takes place one after the other.

The side rail can then be lowered by applying slight pressure from above.

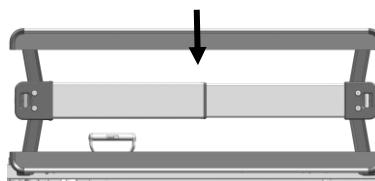
When lowering the side rail, make sure that there are no objects between the moving parts.

The side rail is now in a lowered position.

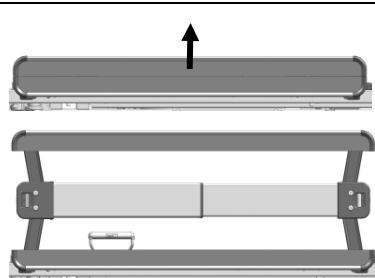To move the side rail to the upper position, grasp the upper side rail bar with one hand and pull the side rail upwards into the upper position. In the uppest position, the side rail audibly engages. The side rail is now in a pulled-up position.

Hazard note from Bock

Please note the increased risk of trapping caused by the raised side rails when adjusting the backrest and upper leg rest.

When using mattresses of different thicknesses, the minimum height of 22 cm, measured from the top edge of the side rail above the mattress without compression, must be reached.

Therefore, the following measurements must be observed for safe use of the side rails:

- Height above the mattress: ≥ 220mm
- Gap measurements between bars and to the lying surface: <120mm
- Gap measurements between side rail and end panel: <60mm or >318mm

2.10 Transport, storage and operating conditions

	Transport and storage	Operation
Temperature	0°C to +40°C	10°C to +40°C
Relative humidity	20% to 80%	20% to 70%
Air pressure	800hPa to 1060hPA	

2.11 Disposal

The individual material components made of plastic and metal are recyclable and can be recycled in accordance with legal requirements. In general, when disposing of the device, make sure that the components to be disposed of are not infectious or contaminated.

3 Cleaning, maintenance and disinfection

By following the care instructions in the main assembly and use instructions, the usability and visual condition of your nursing care bed and side rails will be maintained for a long time. The chapter "Cleaning, care and disinfection" for your nursing care bed applies to the side rails.

4 Regular inspections with service

Regular inspections facilitate the maintaining of the highest possible safety level, and are considered to be an important safety precaution. Medical devices must be inspected regularly in terms of safety according to the stipulated regulations of the manufacturer and the generally accepted rules of technology. The chapter "Regular inspections with service" of your nursing care beds applies to the side rails.

Hermann Bock GmbH

Nickelstr 12

D-33415 Verl, Germany

Tel.: +49 52 46 92 05 - 0

Telefax: +49 52 46 92 05 - 25

Internet: www.bock.net

E-Mail: info@bock.net

Our SALES PARTNERS

Our business partners pursue the same strategy as we do: quality, innovation and above-average standards that are internationally recognised. You can rely on our business partners as you can rely on us.

Please note that training, spare parts supply, repairs, inspections and other services can only be guaranteed by our authorised personnel and our sales partners. Otherwise, all warranty claims will be void.

A listing of our current distributors can be found under www.bock.net/kontakt/vertriebspartner

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