



Quick user's manual domiflex, domiflex classic low, domiflex low, domiflex 185, domiflex 3, adiflex 220

## Hand control

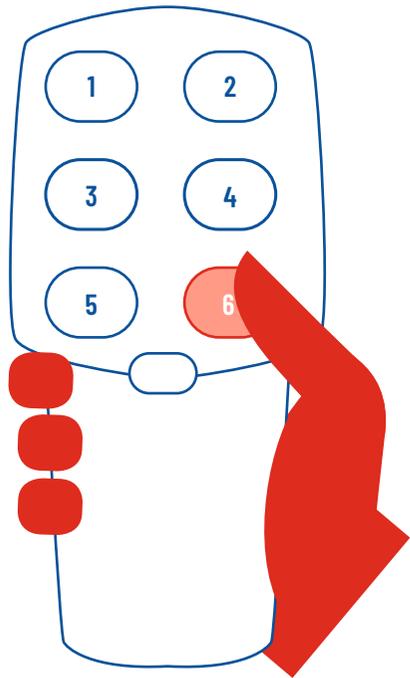


- 1  Back part upwards
- 2  Back part downwards
- 3  Lower leg part upwards
- 4  Lower leg part downwards
- 5  Lying surface upwards
- 6  Lying surface downwards

### Lock function

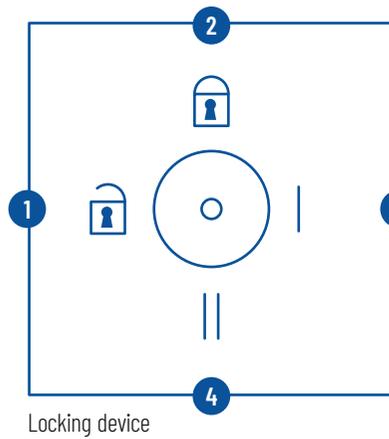

- 1, 3, 4:  
All functions enabled
- 2:  
All hand control functions disabled

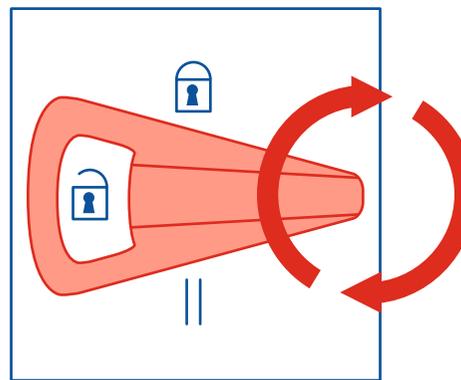

Key for the locking device




.bock<sup>///</sup>



## Castors

To fixate the bed at a place, use your foot to move the locking levers downwards on all four castors.

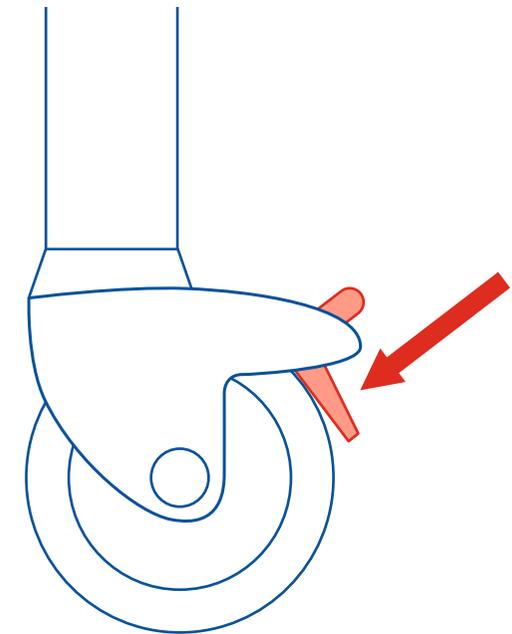

### Important notice:


*This quick user's manual does not replace the assembly and operation manual of the nursing care bed. Please read the operation manual carefully and pay attention to the safety instructions! Thank you.*

Quick user's manual domiflex, domiflex classic low, domiflex low, domiflex 185, domiflex 3, adiflex 220


.bock<sup>///</sup>



## Side rails

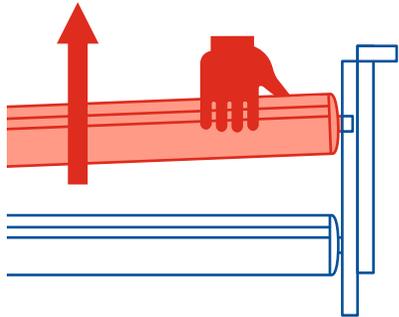


**Step 1: Grab the side rail and lift it up slightly.**

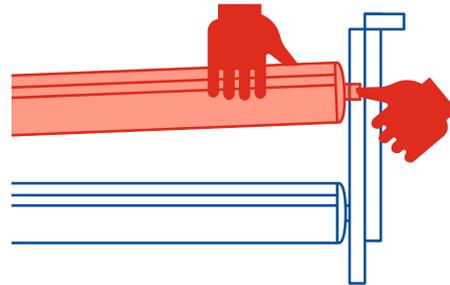


**Step 2: Press the release button.**

Please note: It is necessary to lift up the side rail slightly before you can press the release button.

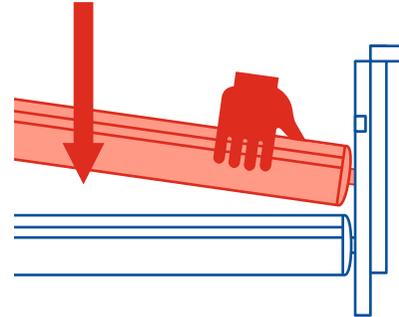

**Step 3: Lower the side rails.**


Attention: Both sides need to be lowered.

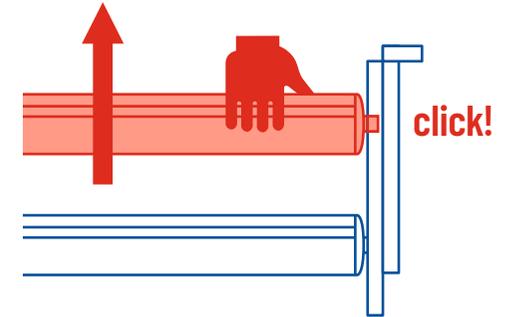

**Upper position as protection against falling:**

Pull the side rails upwards until it audibly clicks into place on both ends.

## Lifting pole

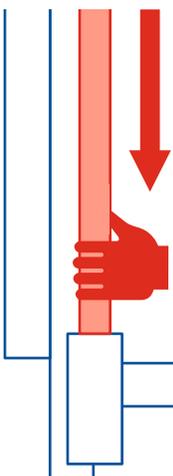


**Step 1: Place the lifting pole in the provided socket at the head part and lock it in place.**


Attention: Do not swivel the lifting pole outside of the lying surface.

**Important note:**

The safe working load of the lifting pole is max. 75 kg.

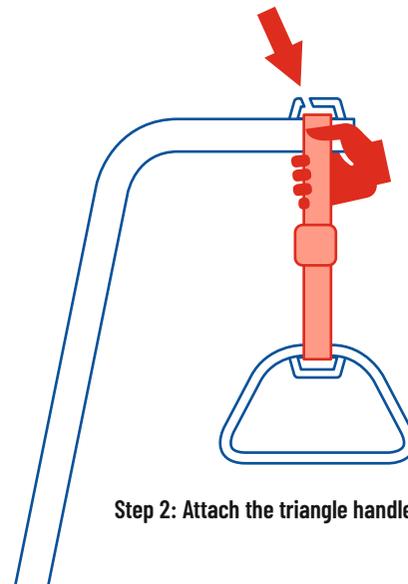

**Step 2: Attach the triangle handle to the hook-up loop.**

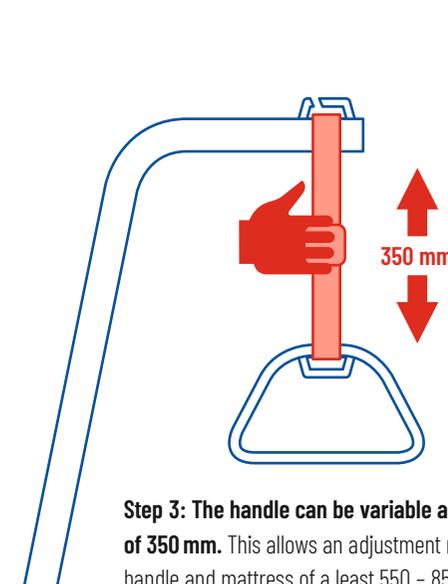

**Step 3: The handle can be variable adjusted within a range of 350 mm.** This allows an adjustment range between triangle handle and mattress of at least 550 - 850 mm.

